



Powerstep®



PROTECH CLASSIC PLUS FULL LENGTH

MATERIALS


- Polypropylene Support Shell
- EVA Base Royal Blue
- Polyester fabric top cover with anti-microbial protection, Navy Blue
- Volara Top Layer, 1/16"
- Intrinsic, cushioning Poron insert at heel
- Full Poron Heel Pad

FEATURES & BENEFITS

- Strong, medical-grade foot support calibrated with flexibility
- Ultra-thin design works in a wide range of shoe types
- Thin, double layer, shock absorbing EVA/Volara cushion casing for enhanced comfort with slim profile
- Friction reducing anti-microbial top fabric
- Deep heel cradle and full Poron heel pad to stabilize and cushion the heel against heel strike

INDICATIONS: Over-Pronation; Plantar Fasciitis/Heel Pain; Foot, Arch and Heel Pain; Calluses; Hammertoes and Claw Toe; Bunions; Neuromas; Ankle and Knee Pain

IDEAL SHOE TYPE: Dress and Casual Shoes, Slip-On Dress Shoes, Low to Mid Heel Dress Shoes, Athletic Cleats

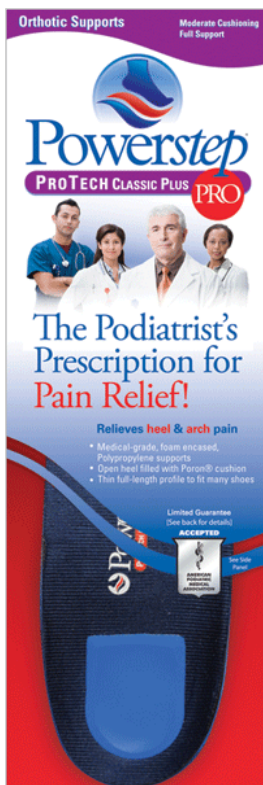

Label Code	Product Code	Size	Men's Size	Women's Size	UPC Code
PC-a	PTCa	Asm	3 ½	5 - 5 ½	749852120121
PC-A	PTCA	A	4 - 4 ½	6 - 6 ½	749852120138
PC-B	PTCB	B	5 - 5 ½	7 - 7 ½	749852120145
PC-C	PTCC	C	6 - 6 ½	8 - 8 ½	749852120152
PC-D	PTCD	D	7 - 7 ½	9 - 9 ½	749852120169
PC-E	PTCE	E	8 - 8 ½	10 - 10 ½	749852120176
PC-F	PTCF	F	9 - 9 ½	11 - 11 ½	749852120183
PC-G	PTCG	G	10 - 10 ½	12 - 12 ½	749852120190
PC-H	PTCH	H	11 - 11 ½	13 - 13 ½	749852120206
PC-I/J	PTCIJ	I/J	12 - 13 ½	14 - 15 ½	749852120213
PC-K	PTCK	K	14 - 14 ½		749852120299

WEARING/FITTING DIRECTIONS: Remove the shoe's existing insoles if possible and replace with your Powerstep insoles. Trim edges if needed to fit. If necessary, wear a few hours a day until feet are accustomed to better support, and then wear full time. Suggest wearing Powerstep insoles in all your shoes, even slippers, to give continuous foot support.

CARE INSTRUCTIONS: Spot clean with a mild detergent. Do not immerse in water. Air dry only. Replace your Powerstep insoles as needed depending on wear.

WARNING: Not intended for persons with severe foot deformities or circulatory disorders resulting in infections or foot ulcerations. Consult your podiatrist or medical professional before using. Always consult a medical professional when experiencing persistent pain, discomfort or swelling.



888.237.3668 · 513.825.1888
Email: info@powersteps.com
www.powersteps.com

Stable Step, Cincinnati, OH
Assembled with pride in the USA!